


Putting on procedure mask with earloops



STEP 1 - Putting on your mask

1. Perform hand hygiene
2. Place mask over your nose/mouth with coloured/printed side facing out
3. Nose wire at top
4. Open the pleats to cover your chin and form the nosepiece lightly across the nose & cheekbones
5. Put your fingers through the earloops
6. Place earloops securely around your ears

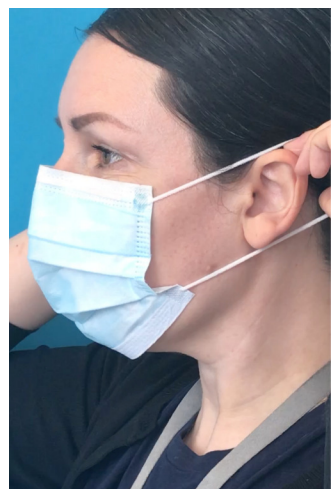

STEP 2 - Forming the nose piece

1. Ensure the mask pleats are opened enough to fully cover your mouth, nose, and chin
2. Gently form (do not pinch) the nose wire over the bridge of nose



STEP 3 - Checking mask fit

1. Make sure the mask is securely fastened around ears and fits well over your nose and chin. There may be some gaps. This mask will not give you a seal
2. Perform hand hygiene



How do I know if the procedure mask is on correctly?

1. The adjustable nose piece is at the top
2. The white (or smoothest side) is on the inside against the wearer's skin—the colour always faces out

Taking off procedure mask with earloops



1. Perform hand hygiene
2. Remove procedure mask by grasping the earloops, pulling out to clear the ears
3. Remove forward, away from your face
Do not touch the front of the mask (if you do, perform hand hygiene)
4. Discard mask
5. Perform hand hygiene



DO NOT

- Leave mask resting on forehead
- Leave mask hanging around neck

